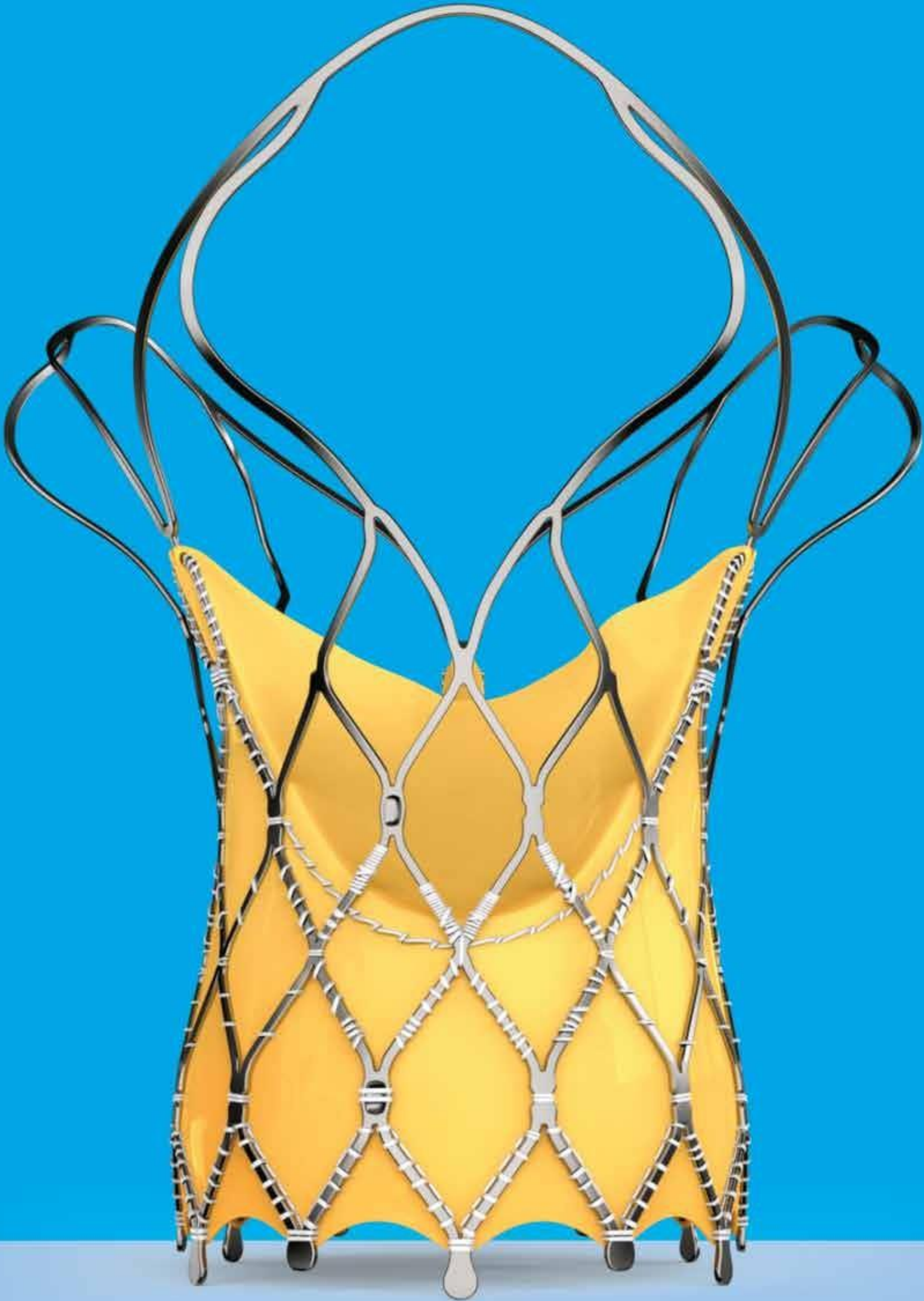
hydra

CE
1434

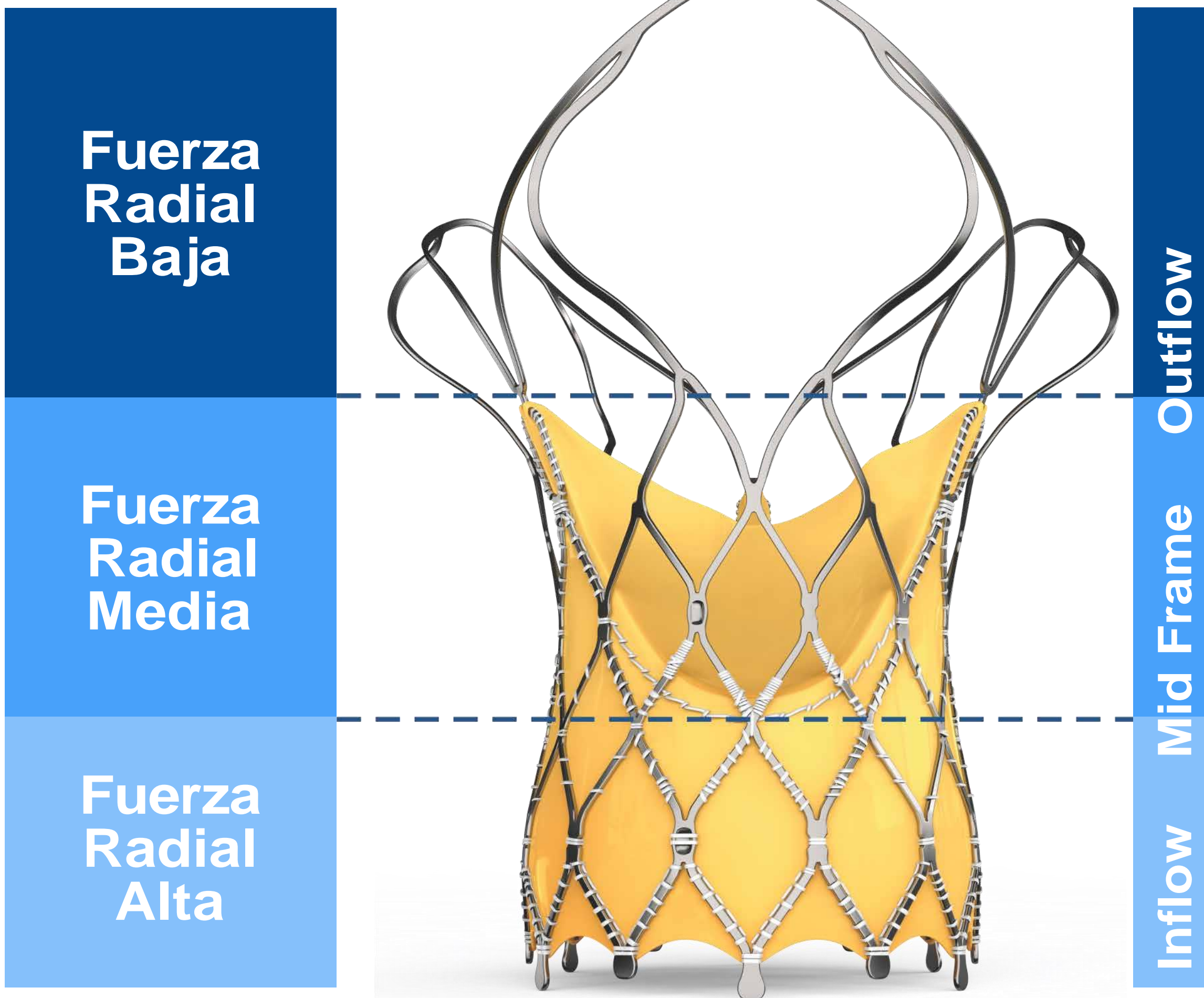
Transcatheter Aortic Valve

Frame material: Nitinol

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
* Static magnetic field of 1.5-Tesla and 3-Tesla, only • Maximum spatial gradient magnetic field of 8,000-Gauss/cm (extrapolated) or less • Maximum MR system reported, whole body averaged specific absorption rate (SAR) of 4-W/kg for 15 minutes of scanning (i.e., per pulse sequence) in the First Level Controlled Operating Mode of operation for the MR system

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

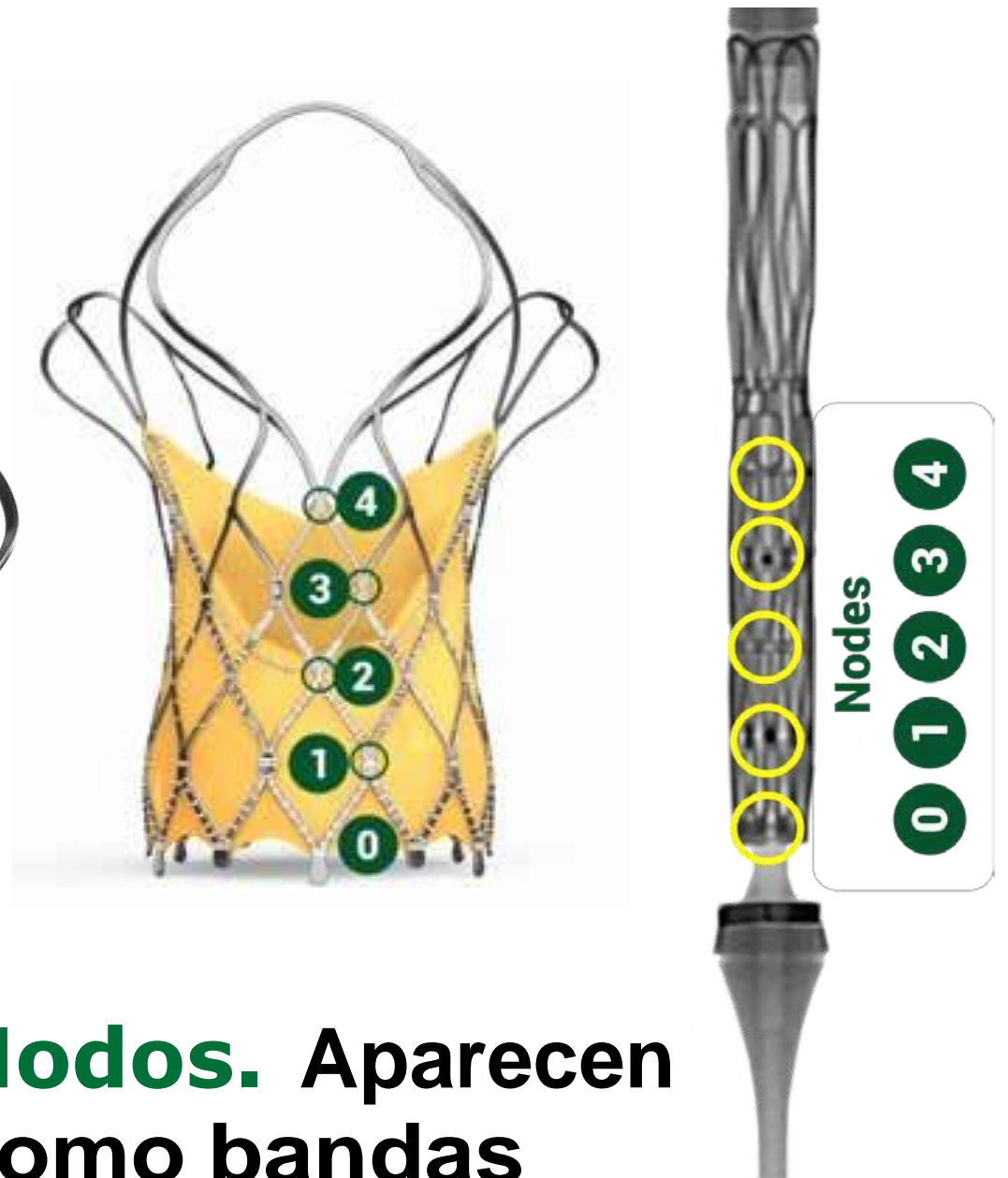
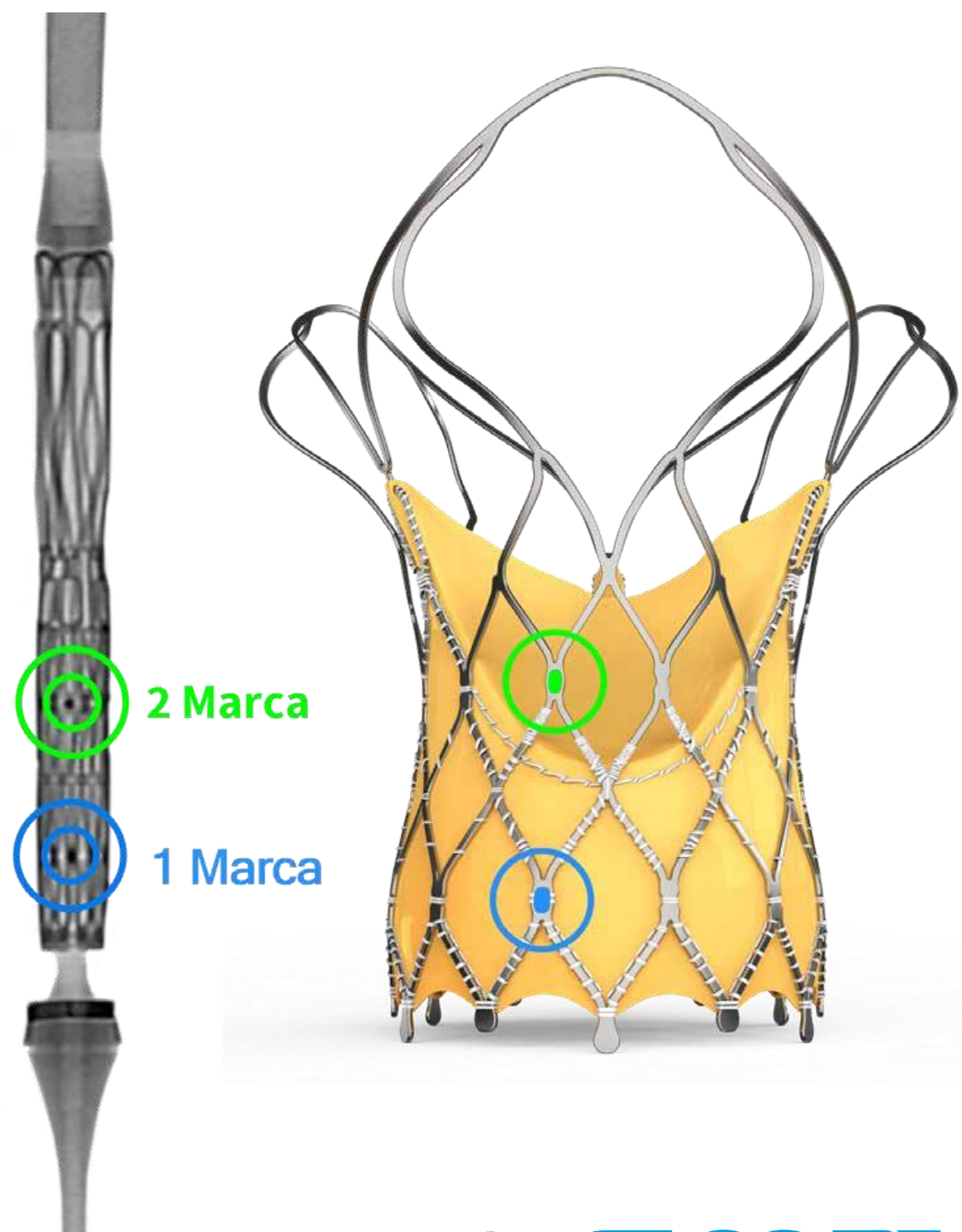
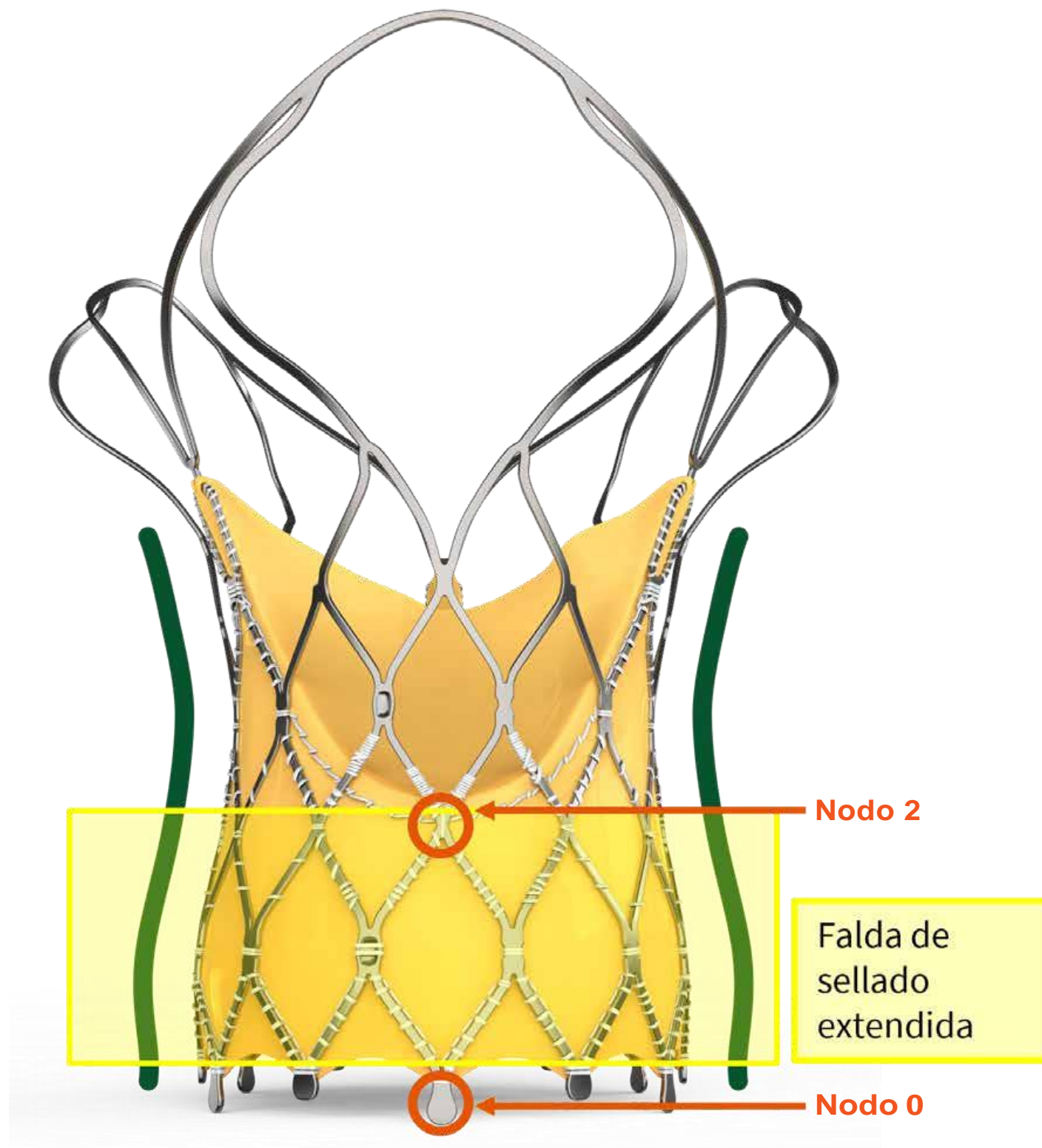
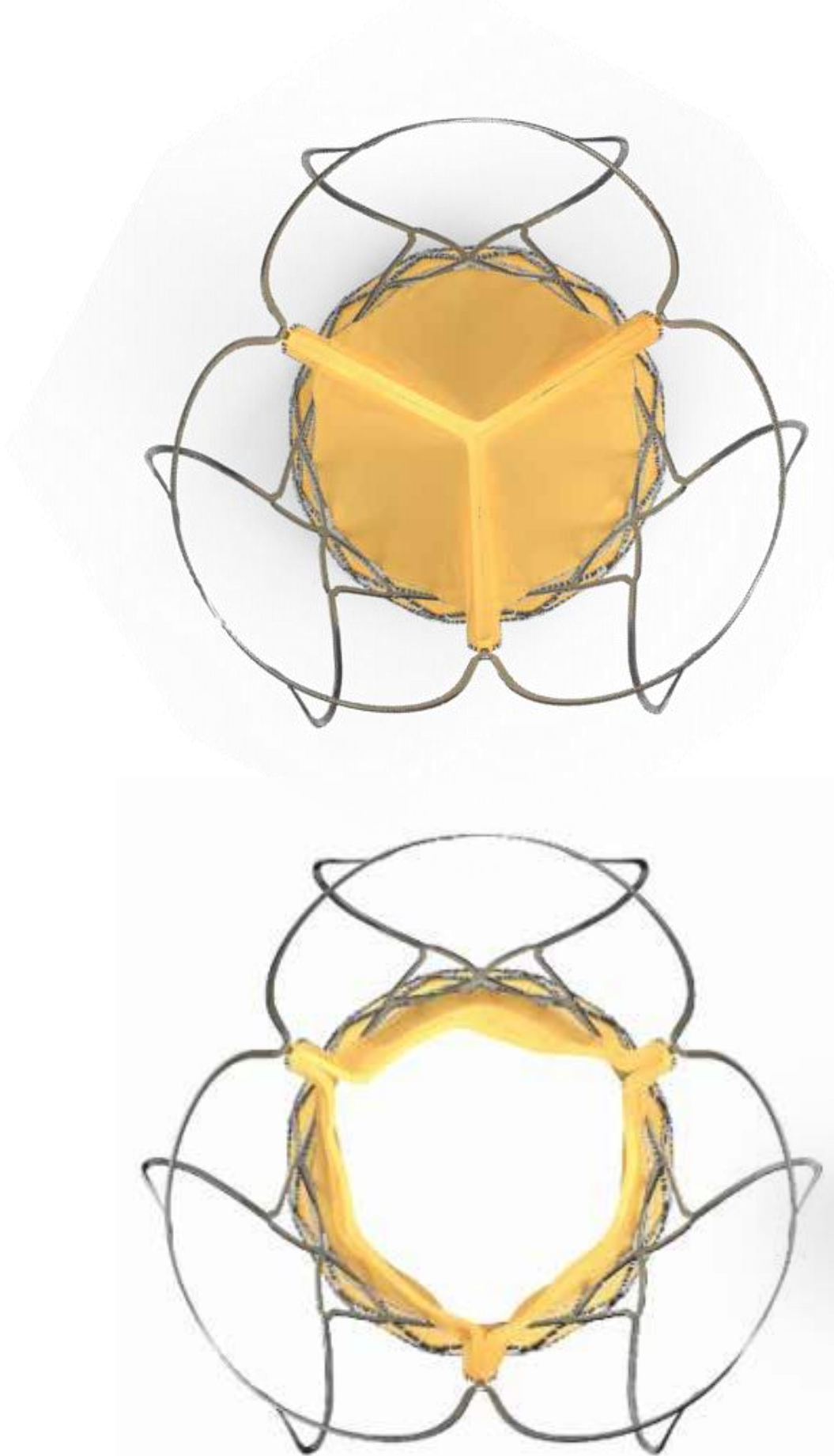
Tamaños Hydra

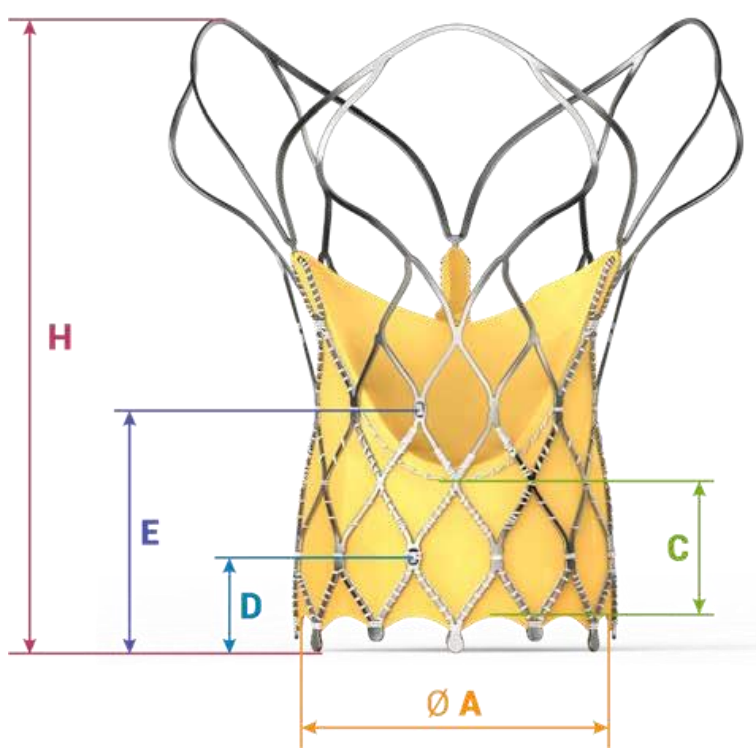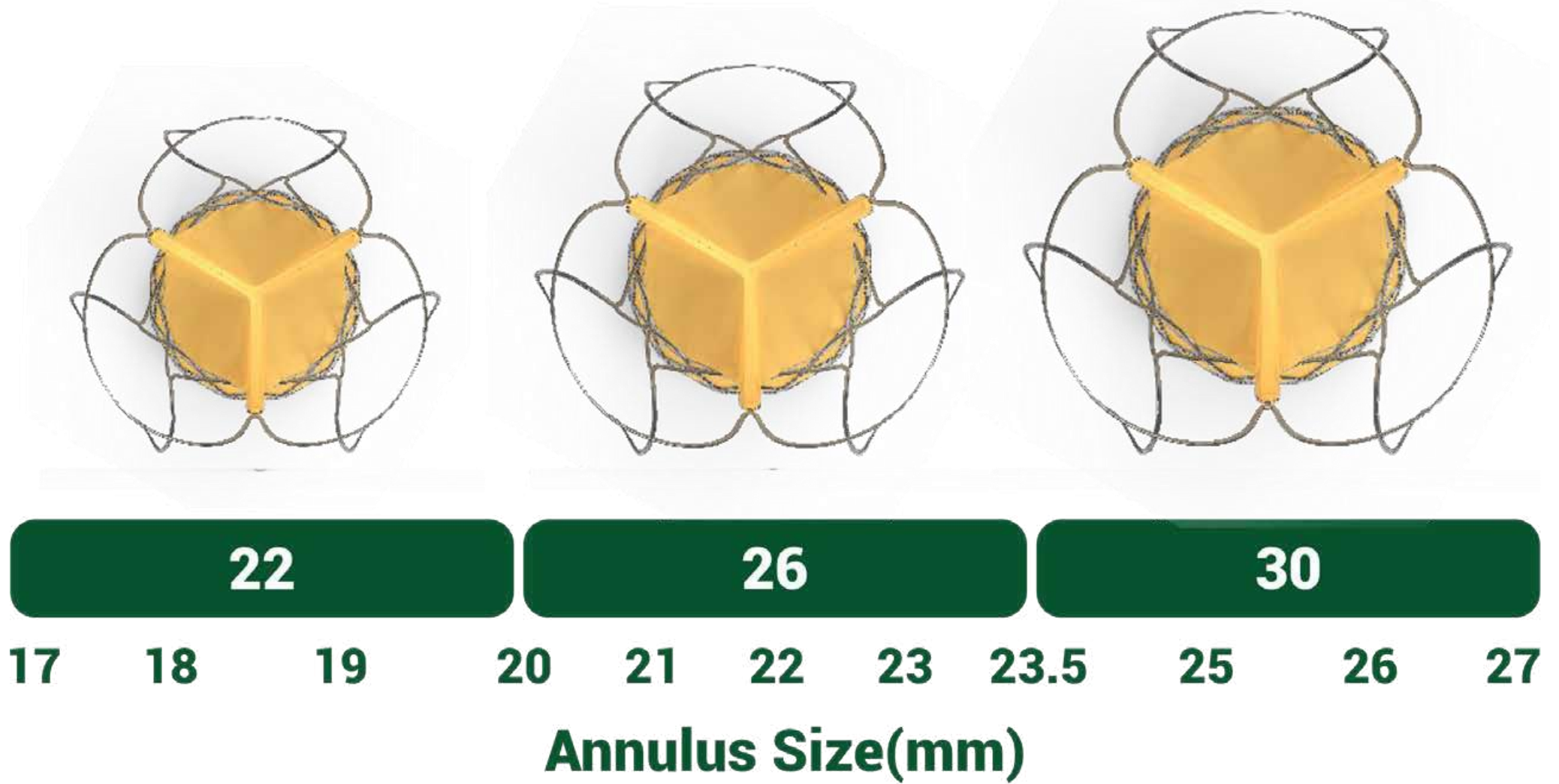Rango de anillos nativos: 17-27 mm

	Hydra 22	Hydra 26	Hydra 30
Diameter at Bottom [A]	22	26	30
Diameter at Top [B]	39	43	47
Sealing Skirt Height [C]	12	13	14
First Marker Crimped [X]	8	9	10
First Marker [D]	5	6	7
Second Marker [E]	15	18	21
Height [H]	55	53	51

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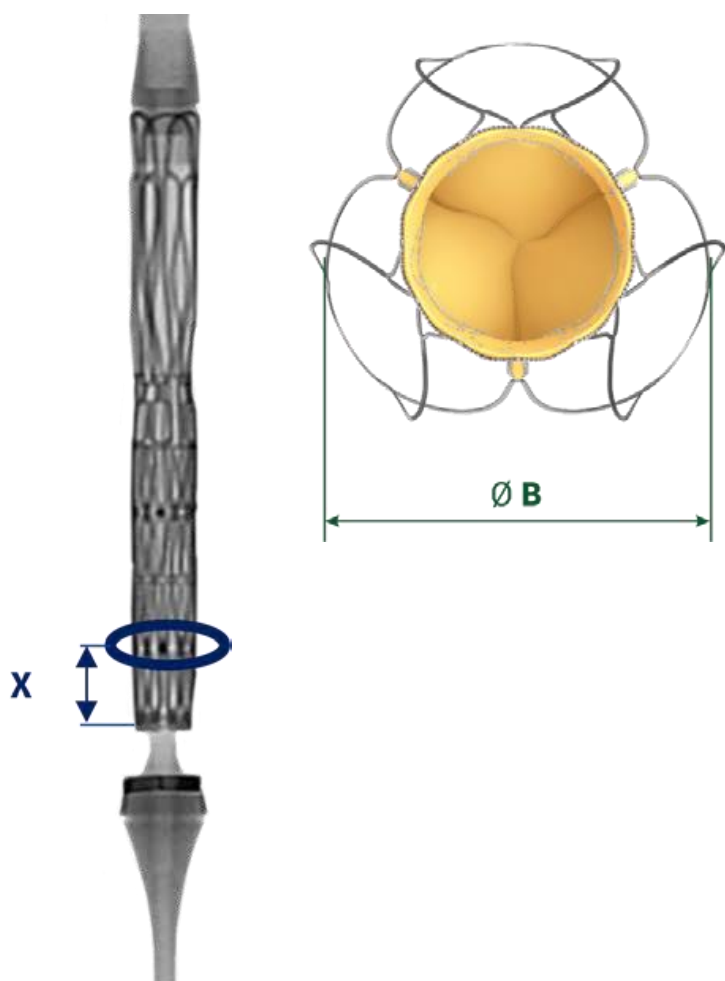*14F equivalent integrated sheath diameter for patients requiring 22 mm or 26 mm Hydra valve.


First-in-Man study (n=15) ¹	GENESIS study (n=40) ²	Hydra CE study (n=157) ³
<ul style="list-style-type: none"> - Published in Asia Intervention - Good hemodynamic performance - EOA increased from 0.68 cm² to 1.53 cm² - Mean Aortic gradient decreased from 49.0 mmHg to 9.4 mmHg 	<ul style="list-style-type: none"> - Published in CCI - 0% Moderate-to-severe-para-valvular leak at 6-month follow up 	<ul style="list-style-type: none"> - Published in JACC : Cardiovascular Interventions - Excellent Haemodynamic - Valve gradient in single digit and remained consistent at 1-year follow up. - Larger effective orifice area up to 1-year follow up. - Low rates of PVL
1 month Follow-up	6 month Follow-up	1 year Follow-up

Ref : 1 https://www.asiaintervention.org/wp-content/uploads/2017/11/web37_AIJ-D-17-00007_Srimahachota.pdf


2. <https://doi.org/10.1002/ccd.29733>

3. <https://doi.org/10.1016/j.jcin.2021.09.004>

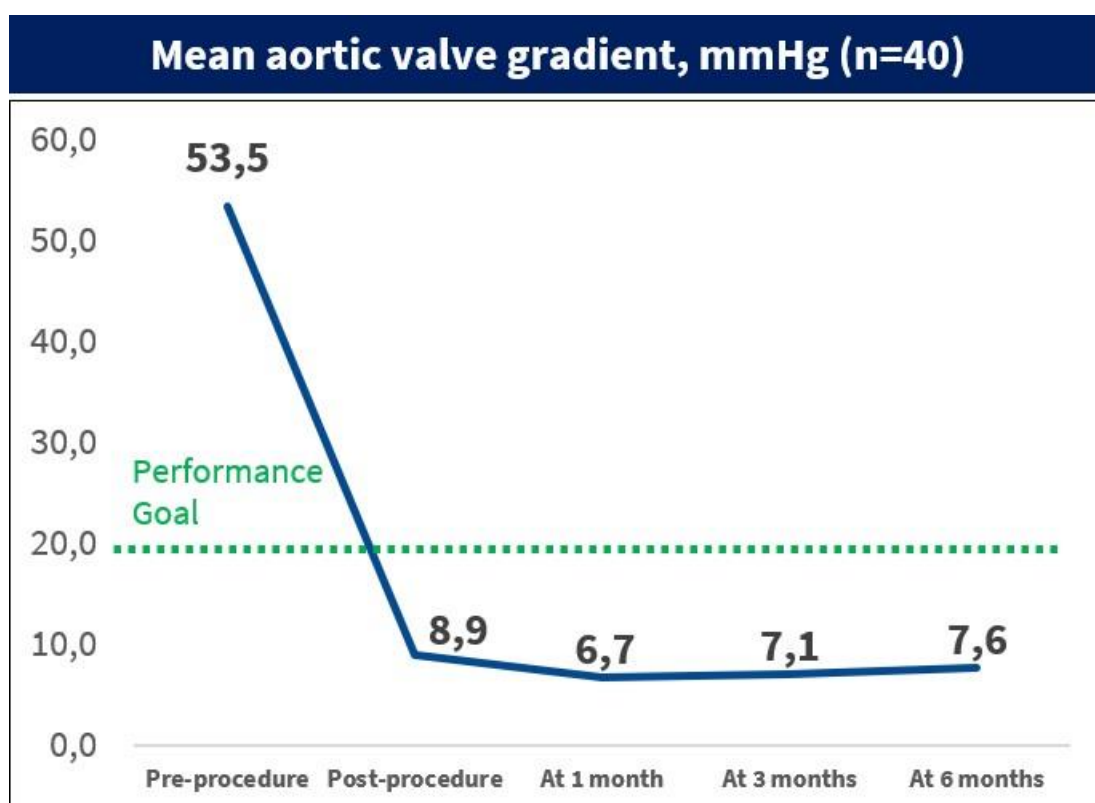Hydra Genesis study: Key Findings

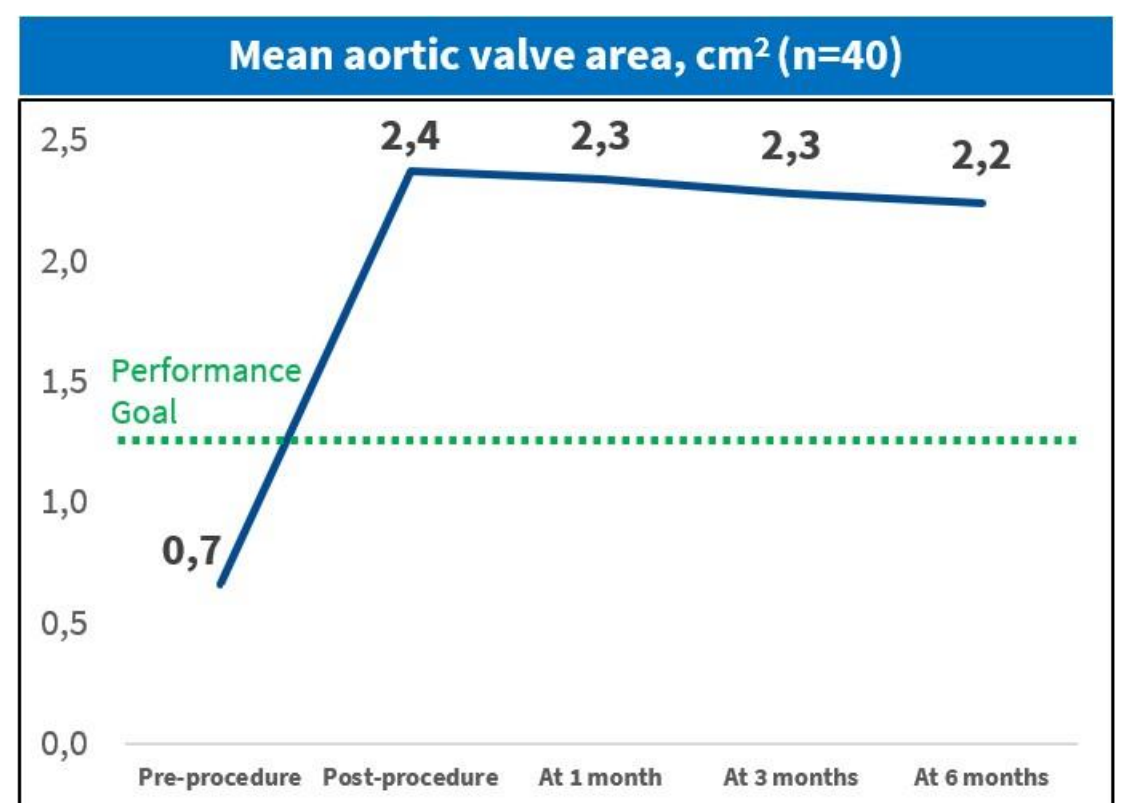Valve gradient was in single digit and remained consistent at 6-month FUP.

Effective orifice area was consistently above 2 cm² up to 6-month FUP.


ORIGINAL STUDIES

Clinical evaluation of the Hydra self-expanding transcatheter aortic valve: 6 month results from the GENESIS trial

Praveen Chandra MD, DM, FACC, FESC, FSCAI, FAPSIC, John Jose MD, DM ✉
Shafeeq Mattummal MD, DM, FESC, FSCAI, FACC ... See all authors ▾

First published: 20 April 2021 | <https://doi.org/10.1002/ccd.29733> | Citations: 4

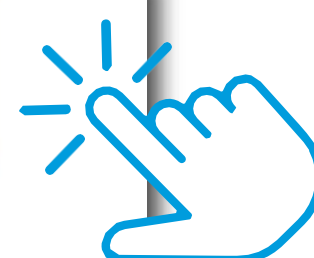Haga clic para acceder al estudio

30-Days and 1-Year Outcomes with HYDRASelf-Expanding Transcatheter Aortic Valve : Hydra CE Study

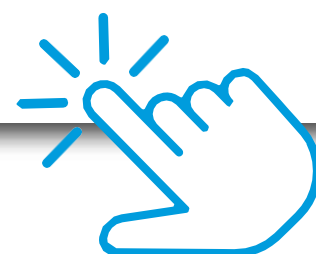
Now Published in

JACC : Cardiovascular Interventions

Impact Factor : 11.195


JACC: Cardiovascular Interventions Vol . 1 5 , No. 1 , 2022 Jan 10 , <https://doi.org/10.1016/j.jcin.2021.09.004>

Haga clic para acceder al estudio


Excellent Hemodynamics

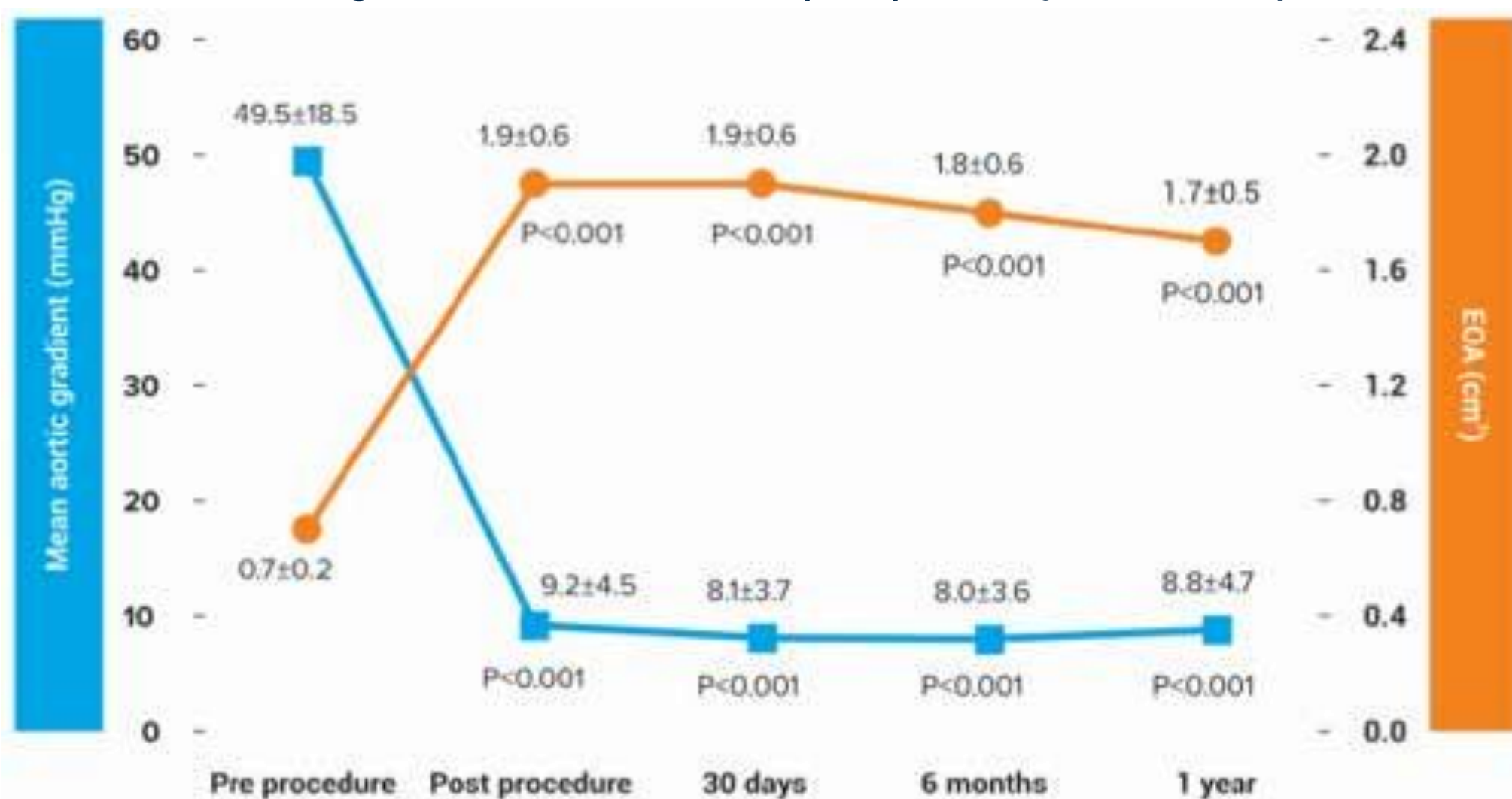Single digit valve gradient up to 1-year follow up
Larger effective orifice area (EOA) up to 1-year follow up

Resultados

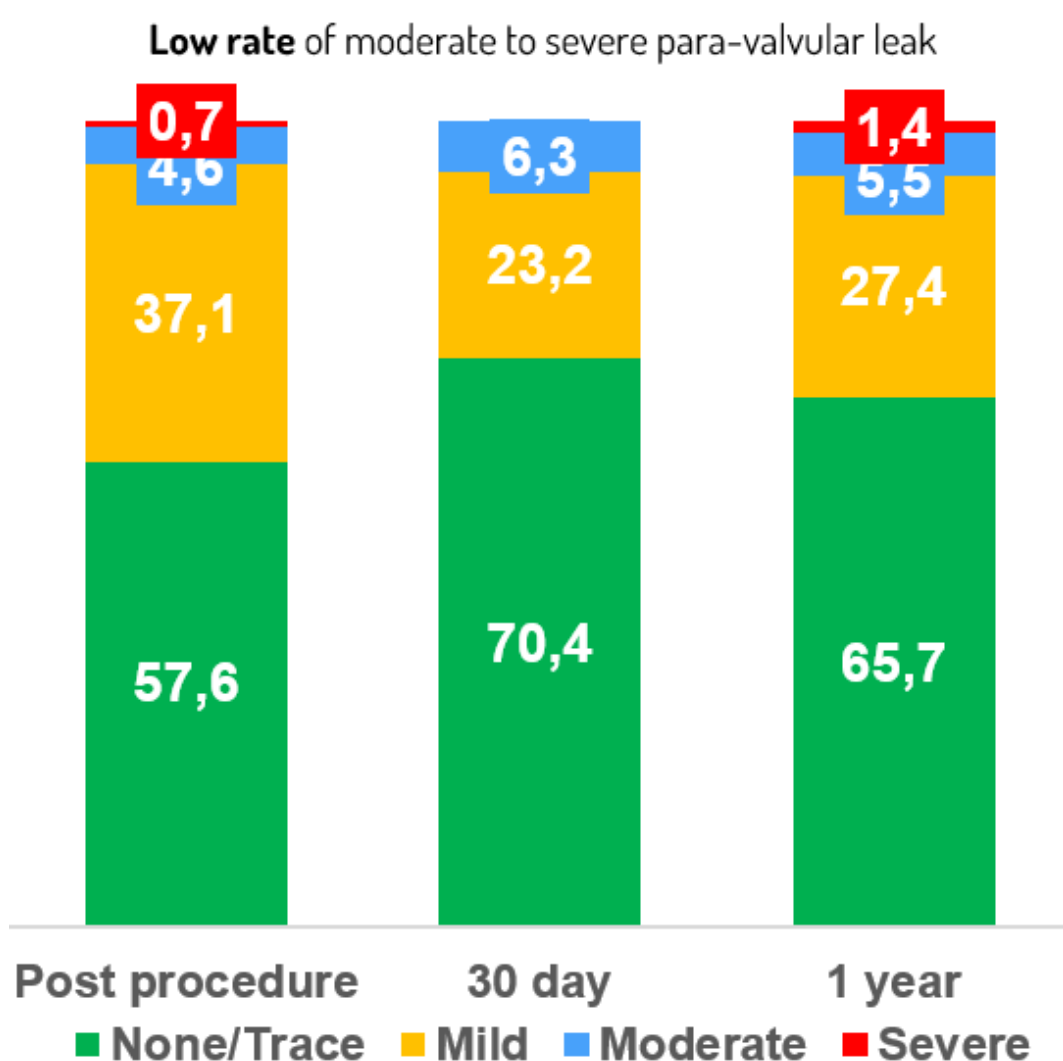
Para-Valvular Leak (%)

NYHA Functional Class (%)

72.4% patients improved by ≥ 1 Functional class

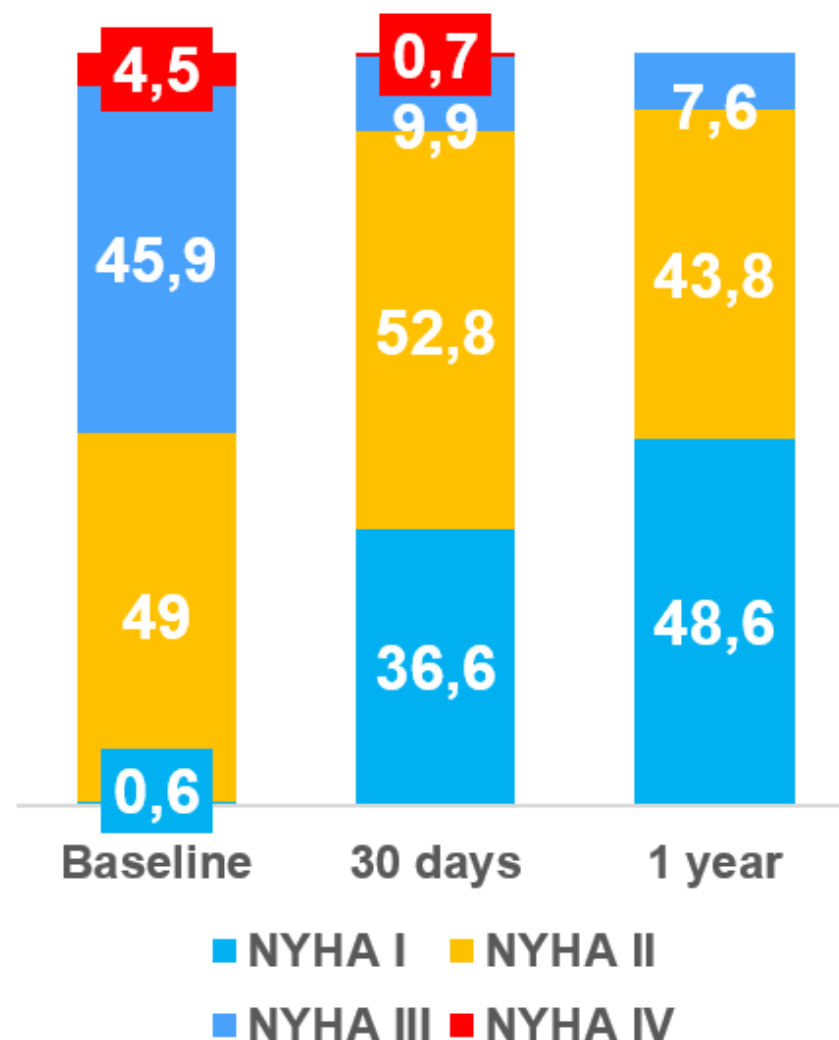21.9% patients improved by ≥ 2 Functional class

JACC: Cardiovascular Interventions Vol . 15 , No. 1 , 2022 Jan 10 , <https://doi.org/10.1016/j.jcin.2021.09.004>
 NYHA: New York Heart Association

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
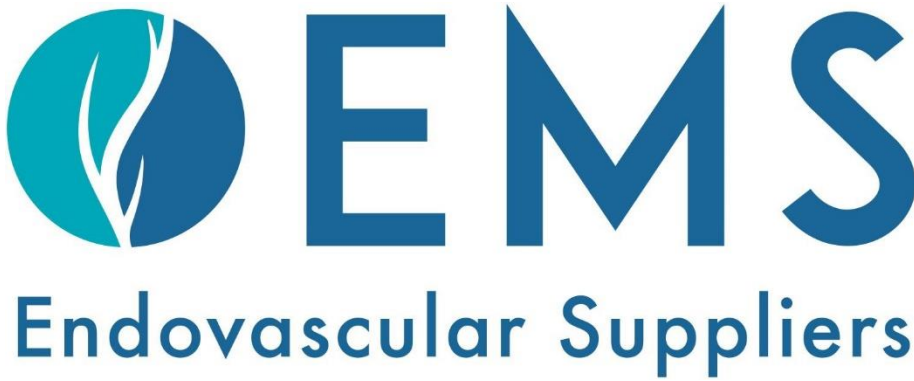
Habla con un representante:

Siga a SMT en las redes sociales

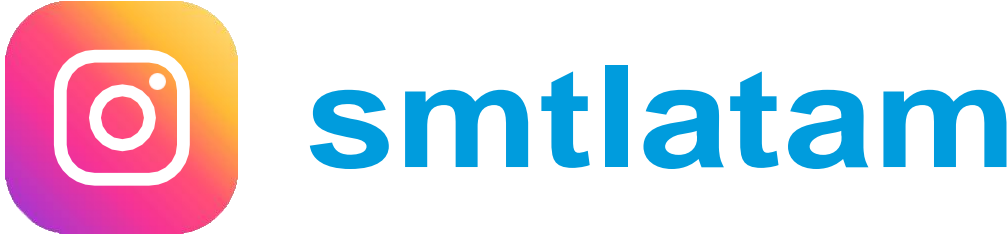Sahajanand Medical Technologies Ltd.
Ground Floor, Block 5, Galway Technology Park,
Parkmore, Galway, H91 R9YR, Ireland.
Tel: +353 9139 7472
E-mail: contact@smtpl.com